

easyCLICK™ Fitting Guide

1 LINK MEASUREMENT

Use the Sebotek Receiver Link Tool to select the proper size of Small, Medium, or Large.


Match the top part of the canal to the corresponding color on the tool for determining the correct size:



Small

Medium

Large

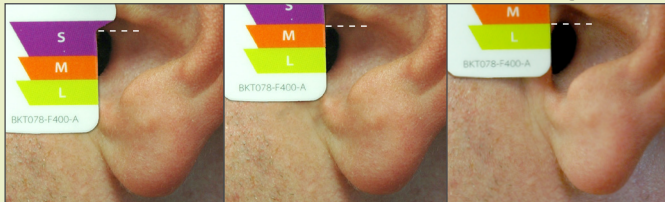


2 PERFORM OTOSCOPY

Do an otoscopy to estimate the size of the patient's canal as either Small, Medium or Large

Match canal size to the following:

Size of Canal	Open Tip	Hi-Fi Tip
Small	8 mm	7 mm
Medium	8 mm	9 mm
Large	10 mm	11 mm

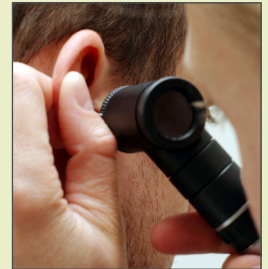

- If feedback occurs, increase size of tip.
- If soreness occurs, decrease size of tip.
- If slipping occurs, install sports lock.

easy  **CLICK**TM